
Quick Start Guide


SmartMat™

GET TO KNOW YOUR SMARTMAT™


TOP VIEW

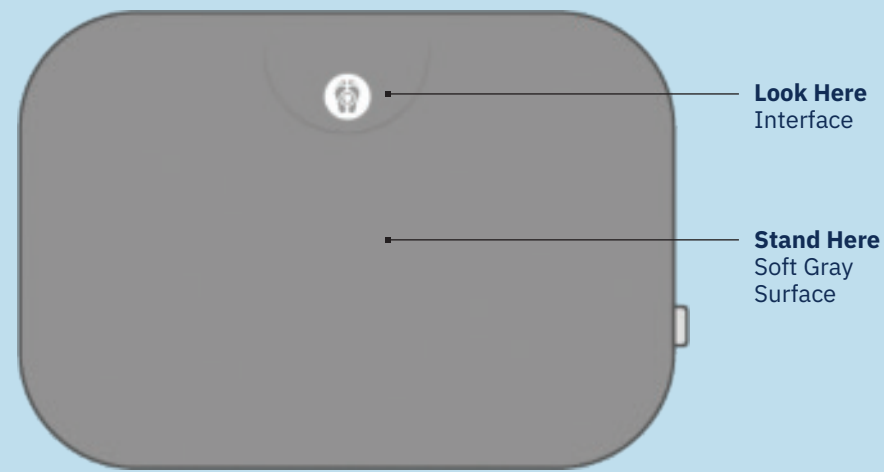SIDE VIEW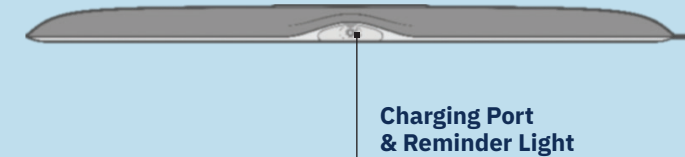

BOTTOM VIEW

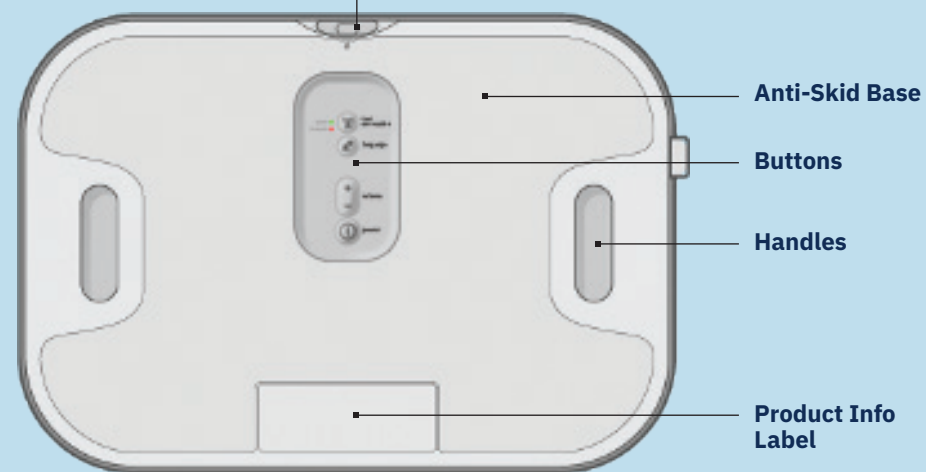
SET UP AND USE YOUR SMARTMAT™

1 UNPACK

Take the SmartMat™ out of the packaging and remove the clear protective cover.

2 SETUP


Turn the SmartMat™ over to access the button pad. Turn the Mat ON, and set the volume and language.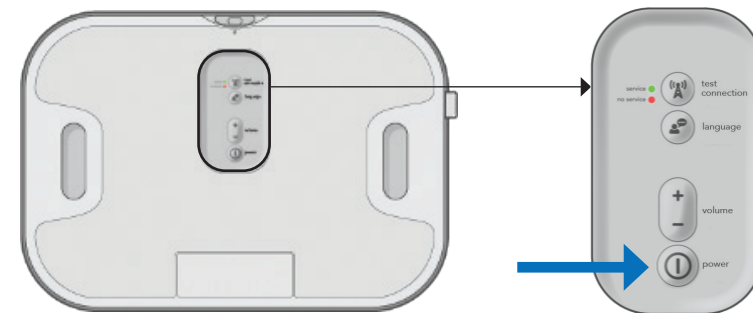

BOTTOM

3 CHECK CELLULAR SERVICE

Make sure the green “service” light illuminates in the place where you’re going to keep your SmartMat™. Press the “test connection” button to recheck.

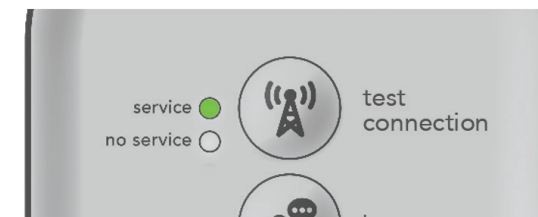4 PLACE THE MAT

Find a flat place for your SmartMat™ without too much foot traffic or water. Put it on the floor with the soft gray side facing up.


5 START A SCAN

Once every day, stand on the SmartMat™ barefoot. Don’t leave it plugged in. The Mat cannot take a scan with a low battery or when plugged in.

6 STAND STILL

Don’t move your feet until the scan is done.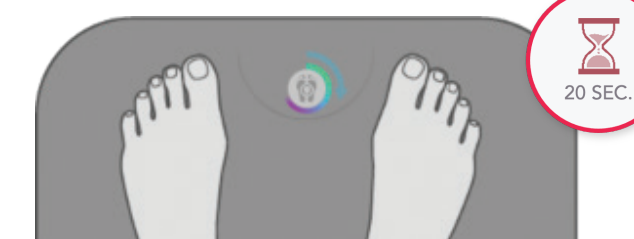


7 SCAN COMPLETE

You can now step off the SmartMat™. Your scan will be automatically transmitted to a server for online review.

TROUBLESHOOTING GUIDE

INTERFACE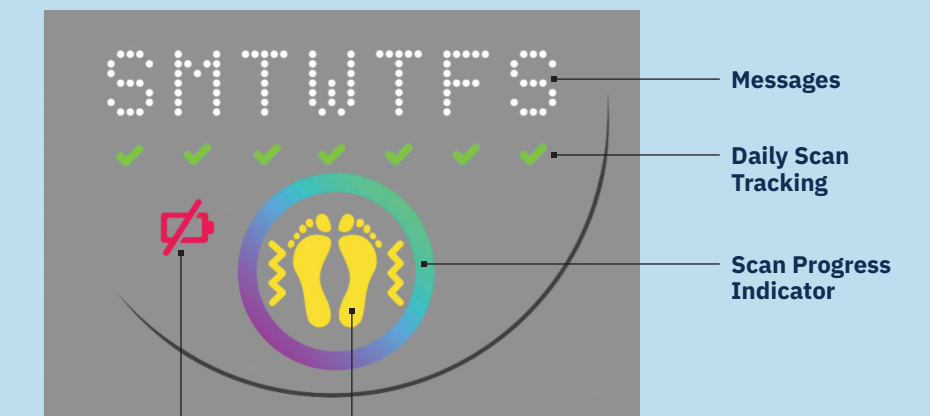


Low Battery Warning
Plug in the provided charger overnight.

Stability Error
Stand still throughout the entire scan. You will see this error symbol if you get off the SmartMat™ before the scan is complete.

BUTTON PAD

No Cellular Service
When you press the “test connection” button, if the red light turns on (no service), move your SmartMat™ to another place with a better cellular signal.

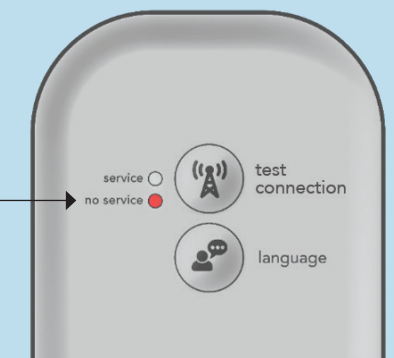QUESTIONS: CALL 1-888-498-6489

Read the User Manual for complete instructions.

WARNING: Do not use the Podometrics Mat with an open wound on your foot.

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